CardioSecur Active

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User Manual ([EN](#))

Manual del usuario ([ES](#))

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Version V7.8

19.06.2023

Index

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Telefon: +49-(0)69-9072013-0
E-Mail: info@cardiosecur.com

Index

1 Additional Information	28
2 Symbols	28
3 Intended Use	31
4 Safety Information	34
5 Connecting the CardioSecur ECG Cable	39
6 Disposable Electrodes	40
7 Serial Number and Contact Address	43
8 Technical Data and Operation Conditions	45
9 Care and Maintenance	47
10 Electromagnetic Compliance (EMC)	48
11 Disposal	51
12 Labelling	51
13 Contact	52

1 Additional Information

This following user handbook describes the usage of CardioSecur Active on iPhone 5/5s/5C, 6/6Plus, SE, 7/7Plus, 8/8Plus and X/XS/XR/XS Max (iPhone is a registered trademark of Apple Inc.) as well as on chosen Android smartphones. The latest list is available for you at www.cardiosecur.com. For reasons of simplicity we refer herein exclusively to smartphone and CardioSecur.

Additional information can be found at www.cardiosecur.com/en/active-manual.

2 Symbols

Packaging included in the scope of delivery or this user manual may display the following symbols (color and size can vary):

Symbol	Meaning
	Use disposable electrodes before the displayed date of expiry
LOT	Lot denomination
	This product complies with the requirements of the directive for medical devices 93/42/EWG.

Symbol	Meaning
	REGULATION (EU) 2017/745 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 5 April 2017
REF	Item number
	Please refer to user manual
	Warning Warning concerns safety-relevant information
	Information Information concerns details that are worth knowing
	Manufacturer's data
	Store at indicated temperature with unbroken seal
	For single use only

Symbol	Meaning
	Latex-free
	CardioSecur and its accessories - like any other electronic device - contain metal and synthetic parts, which need to be disposed of separately after operating life in order to prevent negative environmental impact. Send CardioSecur and its accessories with sufficient postage, and if possible in its original packaging, to Personal MedSystems GmbH for disposal free of charge.
	Application unit of type BF. An application unit of type BF grants protection against electric shock, in particular with regard to permissible discharge current, according to the requirements of norm EN 60601-1.

3 Intended Use

Intended purpose

CardioSecur is intended to record, display and store resting electrocardiograms (ECG) for adults.

CardioSecur is a medical electronic system, comprising an ECG cable and an app for a smartphone. CardioSecur can record, evaluate and store a complete 15-lead ECG using 4 electrodes.


CardioSecur is able to detect arrhythmias and ischemia of the heart by comparing a reference reading to a control reading. CardioSecur serves the purpose of self-monitoring heart activity.

Information for use

The user is required to record a personal reference reading when symptom-free. The reference reading is stored in the CardioSecur app. The user may perform the reading himself or under medical supervision. A control reading may be performed immediately in the case of symptoms or at regular intervals (e.g. every other day), independent of symptoms.

The user is not required to have previous medical knowledge or special training to use CardioSecur. CardioSecur is recommended for individuals with a known or suspected heart condition or risk factors for cardiac disease (e.g. hypertension, diabetes mellitus or increased cholesterol levels). Patients and individuals taking precautionary measures may use CardioSecur.

In order to record an ECG (reference and control ECG), disposable electrodes need to be positioned on the user's thorax according to the following diagram:

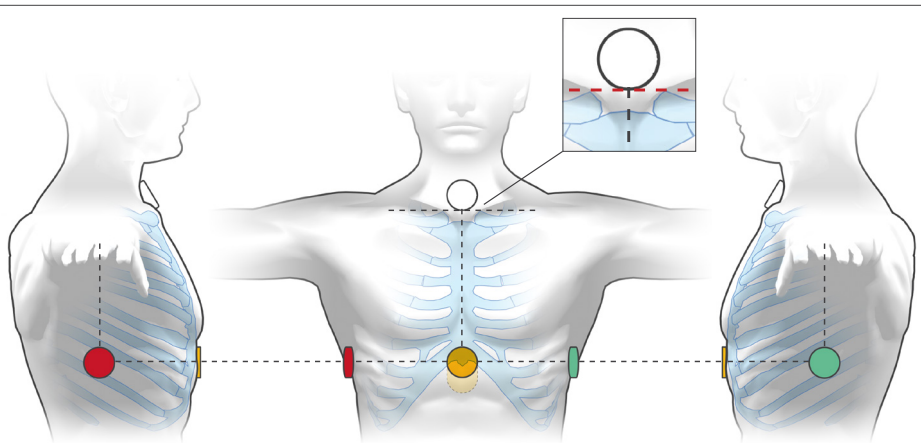White electrode: Position the white electrode precisely in the depression above the breastbone, never below the collarbone level.

Yellow electrode: On the lower, bony end of the sternum or slightly below.

Red electrode: In the middle of the right axillary line at the same level as the yellow electrode.

Green electrode: In the middle of the left axillary line at the same level as the yellow electrode.

Prior to a reading the user is required to sit calmly and relax for a minimum of 3 minutes. During a reading (duration ca. 1 minute) the user must continue sitting calmly and relax without talking or coughing.

CardioSecur performs a comparative evaluation between the reference reading and a control reading and thereupon provides the user with a recommendation to act:

- **White result:** Indicates “no changes” compared to the personal reference ECG. There is generally no need for action.
- **Yellow result:** Indicates “minor irregularities” compared to the personal reference ECG. CardioSecur recommends planning a doctor’s visit for clarification.
- **Red result:** Indicates “severe irregularities” compared to the reference ECG. CardioSecur recommends seeing a doctor instantly.

In accordance with applicable guidelines of the European Society of Cardiology, CardioSecur recommends visiting a doctor instantly whenever symptoms or discomfort prevail for more than 20 minutes. This recommendation applies even when the result given by CardioSecur shows “no changes” or “minor irregularities” compared to the reference ECG.

The user may store, print and manage all ECG readings (reference and control readings) taken with CardioSecur by means of a personal user account on the CardioSecur database, an optional service by Personal MedSystems. In addition users may self-select a physician to view personal ECG data for informational purposes.

4 Safety Information

Take note of the following warnings and information. In case of non-compliance you run the risk of causing malfunction, damage or bodily injury.

Warning: Only use CardioSecur in accordance with its intended use (see [section 3](#), Intended Use).

Warning: CardioSecur contains personalized data. It must only be used by the individual whose personal reference ECG is stored.

Warning: The results provided by CardioSecur are not to be used as the basis for beginning or changing therapeutic measures without an independent medical examination.

Warning: ECG readings by CardioSecur are not able to detect all changes indicative of cardiac disease! If you notice changes to your state of health, contact your physician instantly. In case of prevailing symptoms for more than 20 minutes, contact your doctor instantly, even if the results provided by CardioSecur show no or minor irregularities compared to your reference ECG.

Warning: Do not attempt to make a diagnosis on the basis of the results and analysis provided by CardioSecur. Always check with your physician. Self-diagnosis may cause a deterioration of your state of health or medical condition.

Warning: Results provided by CardioSecur reflect the moment of a recording. Your state of health may change rapidly. In case you notice a change in your state of health, contact a physician.

Warning: CardioSecur serves the purpose of detecting new ischemia and arrhythmias of the heart. CardioSecur is not designed for continuous ECG monitoring, but merely evaluates a short time window of your heart's activity.

Warning: Note that a physician may only receive your ECG recordings and your attached comments if you are actively linked to the physician through the CardioSecur database. Transmission of ECG data to the linked physician is not guaranteed. In addition, when your ECG data is received by your physician, this does not guarantee that he or she will look at the data or establish a diagnosis. It is recommended that you contact the physician you are linked to directly. This is especially important in the case of an emergency.

Warning: Before performing a reading with CardioSecur sit calmly for a minimum of 3 minutes. Ensure that your posture is relaxed. Being seated with a straight back is ideal. Do not cough or speak during a reading. Body movements may cause interference influencing the evaluation.

Warning: Do not perform readings when subjected to vibrations, e.g. in a moving vehicle, train or plane.

Warning: Do not perform ECG readings if the attachment points of the electrodes on your skin are wet, e.g. due to sweat or after bathing.

Warning: Proper functioning of the disposable electrodes is influenced by large amounts of body hair, cream or oily skin. Regularly remove hair from your skin at the attachment points of the electrodes and ensure that your skin is free of cream and fat. Do not clean your skin with alcohol.

Warning: Do not record a reference ECG with incorrectly positioned or transposed electrodes, as a correct evaluation of subsequent control ECGs is not possible.

Warning: Renew your reference ECG after every acute event, intervention (e.g. heart attack, cardioversion in case of arrhythmia, heart bypass intervention, heart pacemaker implantation, cardiac catheter intervention, heart valve intervention), medication change or when recommended by your physician.

Warning: Only use the disposable electrodes recommended by Personal MedSystems (F9067/4/CARDIOSECUR, YD43-6-W, P-00-S/xx or R-00-S/xx).

Warning: Only use disposable electrodes that are unused, have not expired and are at room temperature, in order to ensure sufficient signal quality and prevent distorted results.

Warning: Remove all disposable electrodes after performing a reading, as prolonged adhesion may cause skin irritation.

Warning: Do not use CardioSecur in case of sensitive skin or allergies. This could cause rash, irritation of the skin or eczema.

Warning: Only use the ECG cable as described in this user manual and provided by Personal MedSystems GmbH.

Warning: Do not use CardioSecur when using an external defibrillator (CardioSecur is not defibrillator proof).

Warning: During use of CardioSecur, do not connect any other device or computer via an adapter to your smartphone and/or to the headset jack of your smartphone.

Warning: Do not perform readings in places exposing CardioSecur to high electromagnetic radiation (e.g. TV sets, monitors, dishwashers, dimmer).

Warning: Do not expose CardioSecur to any electrostatics. Discharge any electrostatics out of your body before touching CardioSecur.

Warning: Do not expose CardioSecur to any physical shocks or vibrations. Do not drop it on the floor or step on it.

Warning: Do not place anything onto CardioSecur, as this may damage the device.

Warning: Do not open, take apart or repair CardioSecur. Do not transform CardioSecur. CardioSecur does not contain any parts that need to be maintained by the user.

Warning: Store and use CardioSecur only under the conditions described under the [section 8](#), Technical Data and Operation Conditions, otherwise proper function cannot be guaranteed.

Warning: Do not sterilize CardioSecur in a steam sterilizer (autoclave) or in a gas sterilizer (ethylene oxide, formaldehyde, ozone etc.).

Warning: Do not use CardioSecur in the vicinity of combustible anesthetics, drugs or pressurized oxygen.

Warning: Only use CardioSecur if it has no apparent damage.

Warning: Note the information in [section 9](#), Care and Maintenance.

Warning: CardioSecur utilizes an ECG method computing a 15-lead ECG from 3 leads read with 4 electrodes. Deviations might arise in comparison to a 12-lead ECG read with 10 electrodes.

Attention: Ensure that date and time in your smartphone are set correctly.

Attention: Contact our customer support if you are unable to perform a firmware update.

Note: CardioSecur is suitable for use with pacemakers.

5 Connecting the CardioSecur ECG Cable

Connect the CardioSecur ECG cable to your smartphone.

A CardioSecur with Lightning connector can be plugged directly into an iPhone with Lightning jack (e.g. iPhone 7).

A CardioSecur with micro-USB connector can be plugged directly into an Android smartphone (e.g. Samsung Galaxy S7).

CardioSecur with USB-C can be connected directly to an Android smartphone (e.g. Samsung Galaxy S9).

6 Disposable Electrodes

Disposable electrodes are the link between your body and CardioSecur. They are of critical importance for the technology's proper function. Personal MedSystems GmbH cooperates with highly specialized manufacturers for disposable electrodes in order to ensure optimal signal quality.

Intended Use

F9067/4/CARDIOSECUR, YD43-6-W, P-00-S/xx and R-00-S/xx are disposable electrodes designed for recording electrocardiograms (ECGs).

Warning: Only use disposable electrodes if the expiry date has not passed.


Warning: Only use disposable electrodes if they have reached room temperature.

Warning: Only store disposable electrodes with an unbroken seal and at temperatures between 10 °C and 25 °C.

Operating Instructions


Use disposable electrodes F9067/4/CARDIOSECUR , YD43-6-W, P-00-S/xx or R-00-S/xx according to the subsequent steps:

1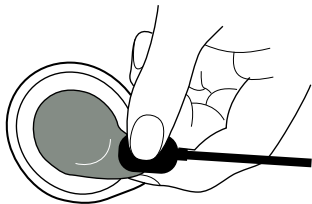

First connect the 4 ECG electrode connectors to an unused disposable electrode.

2

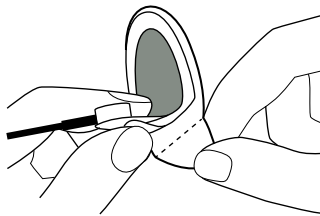Remove the protective foil from the disposable electrodes and attach them without touching the adhesive side and without putting pressure on the middle part (see [diagram 3](#), Intended Use).

3

Remove the disposable electrodes carefully from your skin after performing a reading. Careless removal may cause damage to your skin. Should you encounter difficulties in removing the disposable electrodes, dissolve the adhesive agent of the electrodes with water. Remaining gel may be removed from your skin with paper tissues or water.

Note the instructions provided on the backside of the disposable electrodes' pouch.

Manufacturer Information

FIAB SpA
Via P. Costoli, 4
50039 Vicchio (FI)
Italy

"manufactured and imported"
F9067/4/CARDIOSECUR

Ambu A/S
Baltorpbakken 13
DK-2750 Ballerup
Denmark

Made in Malaysia
P-00-S/xx
R-00-S/xx


Yancheng Tianrun Medical Instrument Factory
Hequiao Industrial Area, West Qingnian Road
Yancheng City, Jiangsu Province
China

Made in China
YD43-6-W


7 Serial Number and Contact Address

1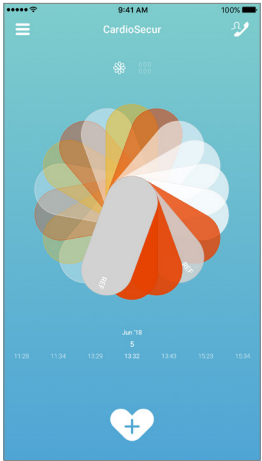

Press the „Menu“ icon (☰).

2

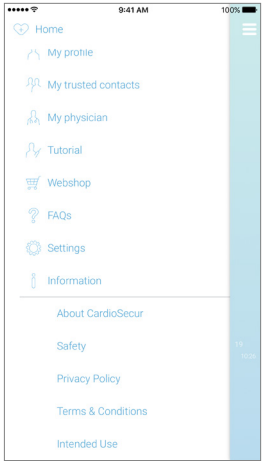Press „Information“ and then “About CardioSecur”.

3

The following information is displayed on the information page:

- Software version of your iPhone/Android app
- Hard- and software version of your CardioSecur ECG cable
- Serial number of your CardioSecur ECG cable
- Personal MedSystems GmbH contact details
- Product certification

8 Technical Data and Operation Conditions

Parameter	Description
Model	CardioSecur Active
Lead method	Bipolar, 3 lead
Reading quality	15-lead ECG with 4 electrodes
Pulse range	18 to 256 beats/minute
Precision	Band width: 0,05 to 40 Hz, sampling rate: 250 Hz
Type of protection	IP22
Storage	10,000 readings
Electrical safety	Application unit type BF
Power supply	through Smartphone
Temperature and humidity operation	+5 °C to +40 °C, 10 % to 93 %, non-condensing
Temperature and humidity storage	-25 °C to +70 °C, 10 % to 93 %, non-condensing
Air pressure for operation	700 to 1060 hPa
Electrodes	Disposable electrodes, type F9067/4/CARDIOSECUR, YD43-6-W, P-00-S/xx or R-00-S/xx

Warning: Do not store CardioSecur under the following conditions:

- Direct sunlight
- Wet or damp places
- Dusty places
- In the vicinity of fire or open flames
- Places subject to physical shocks
- Places subject to high electromagnetic radiation

Information:

- Keep the original packaging of CardioSecur and its accessories in order to identify the product in the case of recall.
- We reserve the right to change technical data without prior notice.
- No safety-related controls are necessary during the product's lifetime.

9 Care and Maintenance

Cleaning and Maintenance of the smartphone

Please follow the instructions from the manufacturer to clean and maintain your smartphone.

Cleaning and Disinfection of CardioSecur

Clean your CardioSecur with a slightly dampened towel. In most cases cleaning with water and soap will be sufficient. Liquids on the casing must instantly be wiped off with a dry towel.

Warning: Never clean CardioSecur with benzene, benzine, paint thinner or concentrated alcohol.

To disinfect CardioSecur, wipe it off with a disinfectant. For disinfecting CardioSecur use a non-aggressive solution. Recommended disinfectants (for superficial disinfection as recommended in the respective manufacturer's user manual) are Helipur® H plus N and Sagrotan® med concentrate.

Warning: Never dip CardioSecur or its parts into a cleaning liquid, and do not sterilize CardioSecur in a steam sterilizer (autoclave) or in a gas sterilizer (ethylene oxide, formaldehyde, ozone etc.).

Maintenance of CardioSecur

Warning: Do not open CardioSecur. CardioSecur does not contain any parts that need to be maintained by the user.

10 Electromagnetic Compliance (EMC)

Warning: Note important information on electromagnetic compliance (EMC). The growing number of electronic devices such as PCs, displays, TV sets and washing machines may subject medical devices to electromagnetic interference. This may cause malfunctioning of the medical device and create potentially unsafe situations. Medical devices should also not interfere with other electronic devices. The norm IEC60601-1-2 was introduced to regulate the requirements for EMC and prevent unsafe product situations. This norm defines the degrees of immunity against electromagnetic interference and the maximum electromagnetic emission levels for medical devices. CardioSecur complies with norm IEC60601-1-2:2001 concerning immunity as well as emissions. Nonetheless, special precautions should be taken. Do not use devices that cause strong electric or electromagnetic fields in the vicinity of CardioSecur. **A minimum distance of 2 meters is recommended. If the distance is less than 2 meters CardioSecur may not perform properly.** In accordance with the norm IEC60601-1-2:2001, contact our customer support (see [section 13](#), Contact) to request further documentation.

DIN EN 550112

Industrial, scientific and medical high frequency devices (ISM devices)
Radio frequency interference – limit values and measuring procedure

Interference emission	Limit value class	Result
Interference voltage	--	n/a
RFI field strength (not accredited testing)	B	OK

DIN EN 60601 1-1-2

Medical electrical equipment

Part 1-2: General requirements for safety:

Collateral standard: electromagnetic compliance; requirements and testing

DIN EN 60601 1-2-47

Medical electrical equipment

Part 2-47: Specific requirements for safety including essential performance characteristics for outpatient electrocardiographic systems

Interference immunity against		Testing accuracy		Result
		normative	tested	
Static electrical discharge	air	8 kV	8 kV	OK
	contact	6 (8) kV	8 kV	OK
High frequency electromagnetic fields (80 – 2500 mHz)	housing	3 V/m	10 V/m	OK
Fast transient disturbances (chest)	mains	2 kV	2 kV	n/a
	I/O	1 kV	-	n/a
Surge voltages	unsym.	2 kV	-	n/a
	sym.	1 kV	1 kV	n/a
Conducted disturbances, induced by high frequency	mains	3 V	10 V	n/a
	I/O	3 V	-	n/a
Energy-related frequency – Magnetic field 50 Hz (Helmholtz coil)	housing	3 A/m	3 A/m	n/a
Voltage dips, short-term interruptions	mains UT	<5 % 0,5 cycles	<5 % 0,5 Per	n/a
Test levels as % of UT duration in supply frequency cycles	100 – 240V 50 / 60 Hz	40 % 100 cycles	40 % 100 cycles	n/a
All testing at 100 V and 240 V each 50 Hz		70 % 500 cycles	70 % 500 cycles	n/a
		<5 % 5000 cycles	<5 % 5000 cycles	n/a

11 Disposal

Warning: Ensure proper disposal of CardioSecur (electronic waste).

Labelling on CardioSecur, its packaging, and this user manual indicate that CardioSecur must not be disposed together with ordinary household refuse after its lifetime. Dispose of CardioSecur separately, so that harm to the environment or human health is not caused by uncontrolled disposal.

12 Labelling

www.cardiosecur.com

Personal MedSystems GmbH
Wilhelm-Leuschner-Str. 41
60329 Frankfurt am Main
Made in Germany 201x

13 Contact

Personal MedSystems GmbH

Wilhelm-Leuschner-Str. 41
60329 Frankfurt am Main
Germany

Phone: +49-(0)69-9072013-0
Fax: +49-(0)69-9072013-99
E-mail: info@cardiosecur.com
Website: www.mobile-ecg.com / www.cardiosecur.com

Refer to our customer support in case of queries regarding CardioSecur.

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© by Personal MedSystems GmbH